



PELAKSANAAN PERTUKARAN SISTEM PACS HCTM



untuk maklumat lanjut:
<https://hctm.ukm.my/radiologi/pelaksanaan-pacs/>

Medweb
SYNAPSE®
SISTEM PACS baru akan mula digunakan mulai
1 JANUARI 2023. Segala kesulitan amat dikesali.
Kesabaran semua amat dihargai




Better . Faster . Friendlier

Panduan ringkas capaian/ paparan imej radiologi menggunakan Sistem C-HeTs -> PACS Synapse.

Langkah 1

1 Akses <http://his.hukm.ukm.my/>

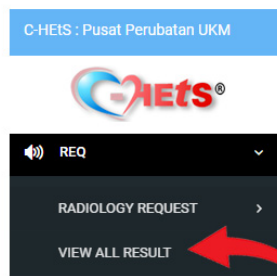

2. Pilih menu REQ - View All Result

Langkah 2

VIEW RESULT

☒ MRN

☐ IC No.

☐ View All Status

A red arrow points to the MRN input field with a green circle containing the number 3. Another red arrow points to the 'View Other Result' button with a green circle containing the number 4.

3. Masukkan MRN pesakit.

4. Tekan atau view other result.

Langkah 3

Status

EXAMINATION

A red arrow points to the 'View Image' button with a green circle containing the number 5.

5. Tekan View Image untuk memaparkan imej radiologi.